

Micro-Z Mini™ Features and Benefits:

- **Conductive Garment electrodes made from silver & Dacron® fibers**
- **Covers the treatment more effectively**
- **Energy, a DC Current from the Micro-Z Mini™ flows from the device into “Intelligent Textiles®” and then into the treatment site**
- **This therapy promotes micro-circulation which relieves pain**
- **Most injuries have inflammation and micro-edema limiting blood flow. By applying a DC current to the treatment area you reduce edema, increase cellular and vascular permeability.**
- **The Micro-Z Mini™ and “Intelligent Textiles® help promote micro-circulation by introducing a very small electrical charge into the cells, tissue and vessels which in turn assists in the movement of blood flow.**

Common Questions:

1. *What is the cost?*

Medicare or Insurance reimbursement is approximately \$350.00 for the Micro-Z Mini™ and \$350.00 for the Silver-Thera™ Stocking “Intelligent Textiles®”(will vary based on DMERC region and private insurer).

2. *How do I use this device?*

Daily, it has a 30 min protocol (P1) which you can do several times a day and/or an 8 hour treatment program (P2). You should try and use the 8 hour (P2) if possible daily and while sleeping.

3. *How quickly will I see results?*

Your personal medical history and longevity with your pain condition determines the effectiveness of the device. Generally speaking, you will have some benefit within the first week or two assuming you use it daily and while sleeping.

4. *How do I clean the stocking (sleeve or sock)?*

Either hand wash or put it on the delicate cycle of your washing machine, but never in your dryer. Based on your use, wash it several times a week.

5. *Will I feel the current?*

Usually not, but you may feel it in the hand. Actually below sensory stimulation is best for the most optimum effect to the body. Too much stimulation overwhelms the cells and body response to this therapy. Increase the stimulation until light sensory stimulation and then

decrease it to below sensory. If you want to feel the current hold it in your hands by touching the two conductive regions.

6. *Why the Thera-Cream™?*

This lotion is 75 times more conductive than water. It has conductive salts and moisturizing cream which prepares the skin to be electricity loving, so that the energy is uniform into the skin from the garment electrodes.

7. *Is this like a TENS device?*

It is a TENS device, but it is different in that DC current delivers the stimulation in the micro amperage range instead of the milli amperage range.

8. *How often do the stockings, socks or sleeves need to be replaced?*

Garments are guaranteed by the manufacturer for 90 days. Based on individual usage, they may need to be replaced after 3 months of use. Washing daily is important for the life and optimum performance of the garment.

Contraindications:

- **Cancer**
- **Pregnancy**
- **Pacemaker**