













## Advanced Operation

PROPRIO FOOT 

Stair Mode Sequence	Hold	Press	BEEP
Stair Mode 6 <sup>0</sup>			3x BEEP
Stair Mode OFF	 REPEAT		1x BEEP
Stair Mode 4 <sup>0</sup>	 REPEAT		2x BEEP

PROPRIO FOOT™ default settings: Stair Mode 4<sup>0</sup>

Relax Mode Sequence	Hold	Press	BEEP
Chair Exit Mode only			3x BEEP
Relax Mode OFF	 REPEAT		1x BEEP
Relax Mode ON	 REPEAT		2x BEEP

PROPRIO FOOT™ default settings: Relax Mode ON

### TROUBLESHOOTING:

Do not fit boots or shoes that cover the ankle area! The PROPRIO FOOT™ will not function properly if ankle motion is restricted in any way.













An alarm and vibratory warning will sound if the ankle gets stuck. Reset the ankle by turning it OFF and then ON.



Life Without Limitations

## Advanced Operation

PROPRIO FOOT 

Function		Hold	Press	BEEP
Power	ON/OFF			2 BEEPS = ON 3 BEEPS = OFF
Set heel height				1x Start 1x Move 1x Complete
Move to neutral/relax				1x Relax
Manual motor movement				N/A
Sound	ON/OFF			1x ON
LGGR	ON/OFF			1x ON/OFF

### NOTE:

*Keypad will be disabled after 60 seconds of inactivity.*

*Turn the POWER ON again to enable the keypad once more.*



Life Without Limitations