



- Faster, Easier & Lighter
- Make a cover in as little as 5 min.
- EZ Cover weighs only 80 g
- Hygienic & easy to revise as needed
- Product No. [4001EZ](#)

- Proprietary thermoplastic mesh
- Special tape designed to bond mesh material and smooth interface
- Preformed shape is easy to adjust by hand without heat


“VOILÀ”

4001 EZ Cover eliminates the need to grind and shape foam covers. Simply adjust size, cut and form. Done!

The 4001 EZ Cover is coated to provide the right balance of rigidity and flexibility.

Fabrication Steps:

1. Adjust EZ Cover to proper circumferential size.
2. Trim the extra portion off with scissors.
3. Attach special tape for fixation onto socket and foot.

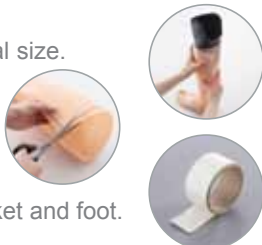

Experience Our Commitment
www.spsco.com | 800.767.7776 x3