ICON

microprocessor knee

college park
TECHNOLOGY *for the* HUMAN RACE

DOMESTIC quick setup guide

kurzanleitung zur einrichtung • guía de configuración rápida •
guide d'installation rapide • guida di installazione rapida

ODYSSEY IQ MICROPROCESSOR FOOT QUICK SETUP GUIDE

NOTICE: The battery is shipped with a partial-charge (up to 30%). We recommend charging it to 100% before use.

PRE-INSTALLATION CHECKLIST

ENGLISH

The Odyssey iQ Microprocessor foot is fully assembled and undergoes electronic testing verification before it is shipped.

WHAT'S IN THE BOX:

HARDWARE:

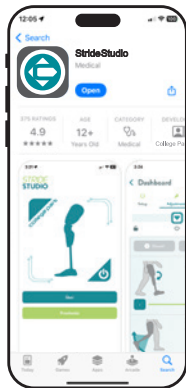
- ☐ Odyssey iQ Microprocessor Foot
- ☐ CPI Sock
- ☐ Foot Shell
- ☐ Battery Charger w/ Adapter

INSTRUCTION MANUALS:

- ☐ Odyssey iQ Microprocessor Foot Quick Setup Guide
- ☐ Odyssey iQ Microprocessor Foot Technical Instructions
- ☐ Odyssey iQ Microprocessor Foot User Manual
- ☐ Stride Studio App for Odyssey iQ Instructions

STRIDE STUDIO APP SETUP

Note: It is strongly recommended that you set up the app and verify Odyssey iQ foot function prior to your patient's appointment.


1. Download Stride Studio app

College Park's Stride Studio App is available for both iOS & Android devices. Download Stride Studio from Apple App Store or Google Play Store.

Minimum Operating Systems:

- Apple iOS 15
- Android Version 8 (Oreo)

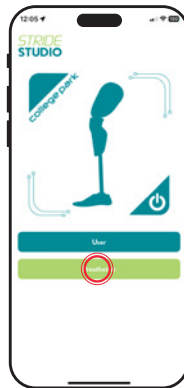Select Stride Studio icon on the device to launch the app.

2. Landing Screen

The landing screen allows you to select whether you are the user or prosthetist

Select "Prosthetist" and you will be taken to the Prosthetist Login Screen.

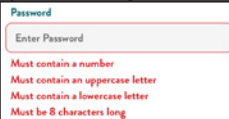
3. Prosthetist Login

When using the app for the first time, clinicians must register for an account. Select “Register”

4. Register

Register for an account by completing the mobile registration form in the app

ODYSSEY IQ SETUP

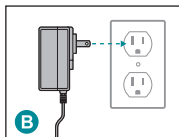
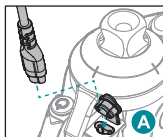
5. Power on the Odyssey iQ Foot

Press and hold the power button for 3 seconds.

6. Charge the Odyssey iQ Foot Battery

The battery is shipped with a partial-charge (up to 30%). We recommend charging the battery to 100% before use.

Only charge the foot on a flat, level surface and in a cool, dry location away from sources of heat and moisture. Plug the wall adapter into the main AC supply **B**.

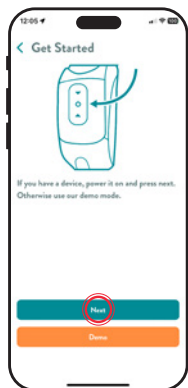
To connect to the front of the ankle, open the dust cover and insert the connector **A**.

Note: The connector is directionally specific.

Note: Ensure charge port is free of moisture and debris before charging.

The foot will audibly indicate when it begins charging or is unplugged.

PAIR DEVICE


7. Enable Bluetooth ✖

Go to Settings on the phone or tablet, then tap Bluetooth.

8. Log into Stride Studio app

Launch the Stride Studio app and log in. The app will prompt you to allow it access to Bluetooth. You must grant it access. You will land on the “Get Started” screen. Select “Next” to pair the Odyssey iQ foot.

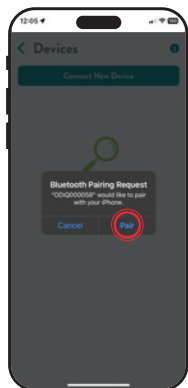“Demo” mode allows you to view the settings without pairing to a foot.

A Bluetooth pairing request will pop up. Press the “Pair” button.

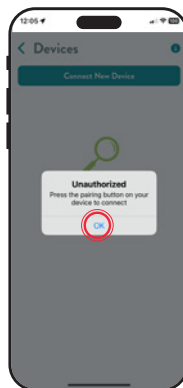
Note: Phone or tablet must be closer than 33 feet (10 meters) of the Odyssey iQ foot.

9. Begin Bluetooth Pairing

The foot must be paired to the Stride Studio app when it is used for the first time. Select “Connect New Device”, and a list of available devices will be displayed. Select the serial number of the foot from the menu.

Note: The foot will be identified by the Serial Number, which is located on the left side of the ankle.

The app will display an “Unauthorized” pop up. Press the power button on the front of the ankle for 1 second to complete the pairing process.

10. If this is your first time setting up an Odyssey iQ foot, the app will play a mandatory training video.

This is a brief overview of the physical setup and programming of the foot. It is also accessible later through College Park Campus.

Note: This video will only play the first time the prosthetist sets up a foot.

11. Congratulations!

The Odyssey iQ foot is ready for patient fitting. Remember to charge the battery fully before delivering the prosthesis.

MEDENVOY SWITZERLAND

Gotthardstrasse 28, 6302 Zug, Switzerland

MADE IN THE USA

©2024 College Park Industries, Inc. All rights reserved.

1171 QS ODIQ 241112

COLLEGE PARK INDUSTRIES, INC

27955 College Park Dr. Warren, MI 48088 USA

EMERGO EUROPE

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

Australian Sponsor

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street,
Sydney, NSW 2000 Australia

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
This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

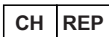This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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