



Liner Washer

6x14 Screw



Manufactured by



419 N. Curtis Rd., Boise, Idaho 83706 (208) 429-0026 | www.coyotedesign.com

CD120L

Lanyard and Low Profile Screw

Fabrication Instructions



2-year warranty against manufacturer defects, excessive wear or breakage.

Patent No. 6334876 Made in U.S.A. External Prosthetic Components



Advena Ltd Pure Offices Plato Close Tachbrook Park Warwick, CV34 6WE, UK



Attaching Lanyard Strap



1 Attaching strap to liner.



Use Loctite 242 on all threads.



Screw adaptor screw in until it is flush with the liner.



4 Use a hot awl or punch to make a hole for 6x14 screw. Screw should fit through hole easally.



Put 6x14 screw and liner washer through hole in the lanyard strap.



6 Use Loctite 242 on 6x14



Screw in 6x14 Screw until it is tight.



8 Attach strap to liner and feed strap through lanyard slot.



9 Use lanyard strap to attach chafe in proper location.



10 Add desired rivet to hold chafe.



11 Cut strap to desired length.