
# SLEEQ

spinal • therapy • system

## INSTRUCTIONS FOR USE

### SLEEQ EXT

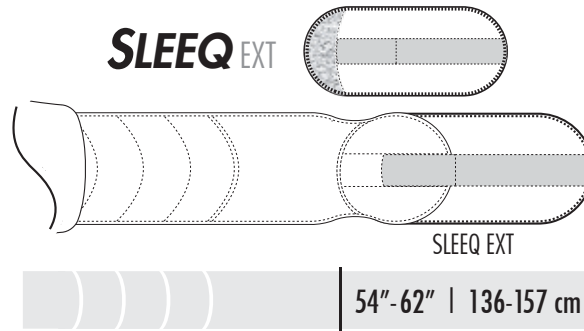#### Care Instructions:

HAND WASH COLD

DO NOT DRY CLEAN

HANG OR LAY FLAT TO DRY

DO NOT BLEACH

DO NOT IRON

## 1.0

### Start



### Empezar

### Start

### Démarrage

### Avvio

#### 1.1

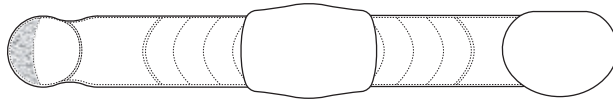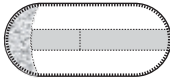#### Place

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
#### 2.2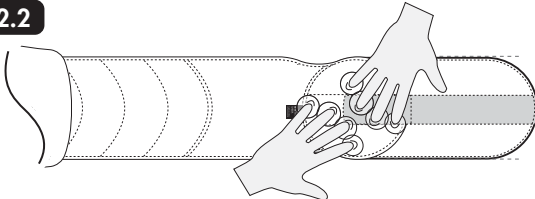

#### Secure

Confirm SLEEQ EXT is firmly secured to Wing on both sides.

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
2.3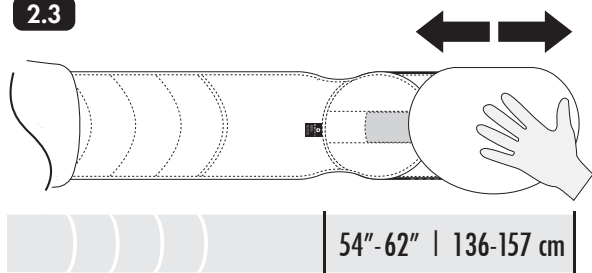

### Re-attach

If included, re-Attach AeroFlex Anterior Panel. This support panel may be moved left or right to ensure it is centered on patient's abdomen.

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## SLEEQ-Anwendung

## Application de la ceinture SLEEQ

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Refer to Instructions for Use included with every SLEEQ brace to apply as usual.

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### LIMITED WARRANTY

Quinn Medical warrants to the original purchaser that this product is free from defects in materials and workmanship. Normal wear and tear during the use of a product is not considered a defect. This Limited Warranty is effective from the date of delivery to the original purchaser.

This warranty does not apply if this product:

- (1) has been altered in any way;
- (2) was not purchased from Quinn Medical or one of its authorized distributors;
- (3) is not used in accordance with Quinn Medical's Instructions for Use (IFU).

Under this limited Warranty Quinn Medical's sole obligation shall be to repair or replace the defective product or associated part(s) at no charge. This Limited Warranty does not cover damage as a result of mishandling, accidents, improper use or alteration of the product. This warranty is in place of all other warranties, either express or implied, including but not limited to warranties of merchantability or fitness for a particular purpose. Quinn Medical shall not be liable under this or any implied warranty for any personal injury, property damage, or special, incidental, or consequential damages arising out of, related, or incidental to the use of the product even if Quinn Medical has been informed of the prospect of such possible loss or damage, unless state law otherwise prohibits this exclusion. User should seek advice from their healthcare provider immediately if user encounters any kind of adverse reaction related to the use of this product. User should consult with their healthcare provider with any questions or concerns relating to the proper use of this product for their specific medical condition. IMPORTANT- Read all Instructions for Use (IFU) and any other associated materials to ensure proper use.

### Warnings

- Carefully read use/care instructions and warnings prior to use.
- If increased pain, swelling, sensation changes, or any adverse reactions are experienced while using this product, immediately consult your medical professional.
- For single patient use only.

### Contraindications:

- SLEEQ braces are contraindicated for women who are pregnant.
- SLEEQ braces are contraindicated for persons with circulatory, pulmonary, cardiovascular or other conditions where increased abdominal pressure would be of concern.

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