







Aspen Infant & Pediatric Collars  
A Proper Fit as Easy as "1 - 2 - 3"

- 1. Chin-Dots hidden?** The chin must be flush with the front of the chin piece plastic.
- 2. Velcro® past mark?** Back panel plastic must overlap the sides of the front panel.
- 3. FlexTabs™ bent?** The FlexTabs™ on the front and back panels should be flexed.

**Sizing Guidelines (Note: New Label Colors)**

PD <b>1</b>		1 - 18 months	
		11 - 29 lbs. <b>weight</b> 5 - 13 Kg.	21 - 33 in. <b>length</b> 53 - 84 cm.
PD <b>2</b>		9 - 24 months	
		22 - 33 lbs. <b>weight</b> 10 - 15 Kg.	29 - 37 in. <b>length</b> 74 - 94 cm.
PD <b>3</b>		1 - 3 years	
		24 - 36 lbs. <b>weight</b> 11 - 16 Kg.	33 - 40 in. <b>height</b> 84 - 102 cm.
PD <b>4</b>		2 - 5 years	
		26 - 42 lbs. <b>weight</b> 12 - 19 Kg.	35 - 45 in. <b>height</b> 89 - 114 cm.
PD <b>5</b>		3 - 6 years	
		27 - 54 lbs. <b>weight</b> 12 - 25 Kg.	37 - 48 in. <b>height</b> 94 - 122 cm.
<b>ADULT Short</b>		5 years and up	
		<b>weight</b> 36 lbs. (16 Kg.) and up	<b>height</b> 42 in. (107 cm.) and up



Please Refer to Pediatric Collar Instruction Sheet for more information.

© 2002 - 2004 Aspen Medical Products, Inc.  
(800) 295-2776 • (949) 681-0200  
www.aspenmp.com


983174 REV J 10/04

Aspen  
CERVICAL COLLAR SIZING GUIDE

**Pick the Right Size**

With the head in the neutral or desired position, place the guide on the highest point of the trapezius and flat against the side of the head.

Sight an imaginary perpendicular line from the bottom of the chin back to the guide and select the proper size collar.

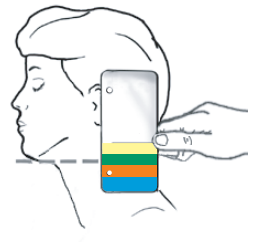

**Check for a Good Fit**

- The chin must be flush with the front of the chin piece plastic.
- The FlexTabs™ on the front and back panels should be flexed.
- The collar should be "snug" but not touch the tracheal area.

**Extra Tall**

**Tall**

**Regular**

**Short**

Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.