PostOp Knee Brace

POSTOPERATIVE KNIESCHIENE
SUPPORTO POST-OP DEL GINOCCHIO
ORTHESE DE GENOU POST-OPERATOIRE
RODILLERA POSTOPERATORIA

PostOp Lite Knee Brace

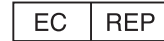
LEICHTE POSTOPERATIVE KNIESCHIENE
SUPPORTO LEGGERO POST-OP DEL GINOCCHIO
ORTHESE DE GENOU POST-OPERATOIRE LEGERE
RODILLERA LIGERA POSTOPERATORIA

PostOp Knee Brace with shells

POSTOPERATIVE KNIESCHIENE MIT SCHALEN
SUPPORTO CON INVOLUCRI POST-OP DEL GINOCCHIO
ORTHESE DE GENOU POST-OPERATOIRE A ENVELOPPES
RODILLERA POSTOPERATORIA CON ARMAZONES

PostOp QuickFit Knee Brace

POSTOPERATIVE QUICKFIT-KNIESCHIENE
SUPPORTO DEL GINOCCHIO POST-OP QUICKFIT
ORTHESE POST-OPERATOIRE QUICKFIT
RODILLERA POSTOPERATORIA QUICKFIT

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AW-1.90032 REV A

WARNINGS

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: FOR SINGLE PATIENT USE ONLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

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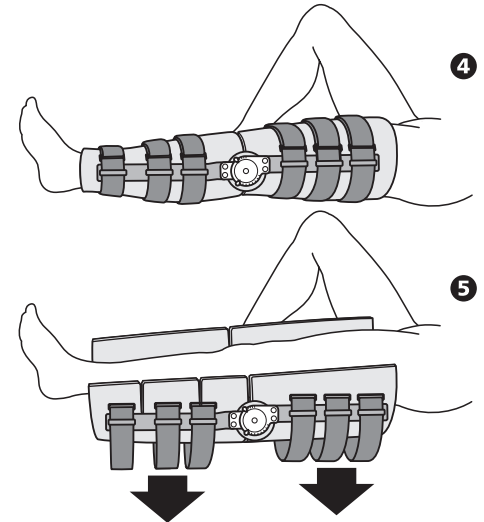
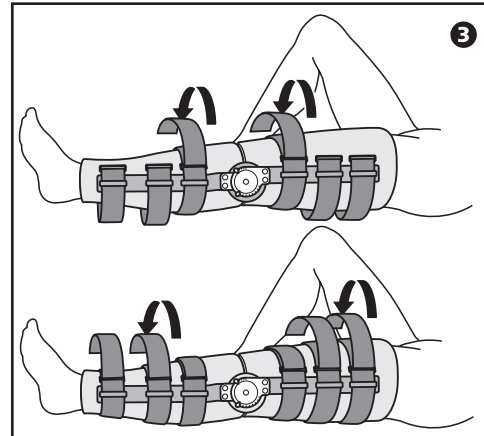
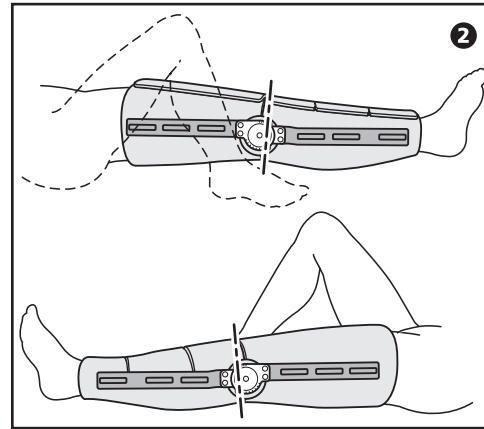
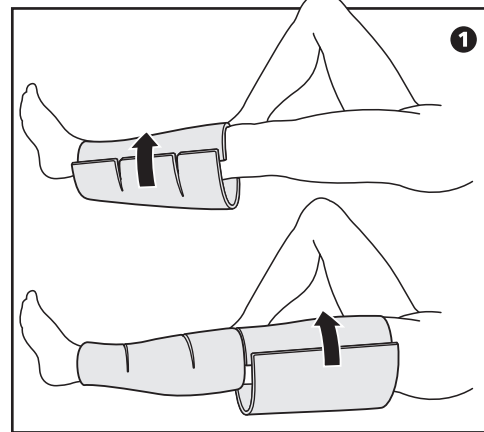
- 1 Wrap and fasten the foam wraps around the leg using the sidebar as a gauge for positioning.
- 2 Position and press into place the side bars. Reposition the wraps if necessary to align with the straps. The hinges should be placed slightly above the middle of the knee cap and toward the back of the knee on the sides. For illustration purposes the straps are not shown in step 2. Do not remove the straps for application.
- 3 Secure and fasten the straps closest to the knee first, then the remaining straps.
- 4 A properly applied POST-OP KNEE BRACE.
- 5 To remove the brace, simply unfasten the straps and open the foam pads. The brace can then be removed and reapplied as a single unit.

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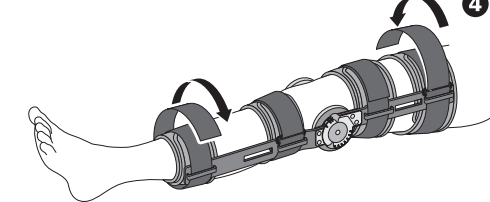
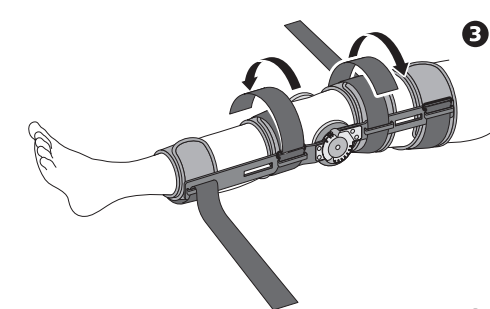
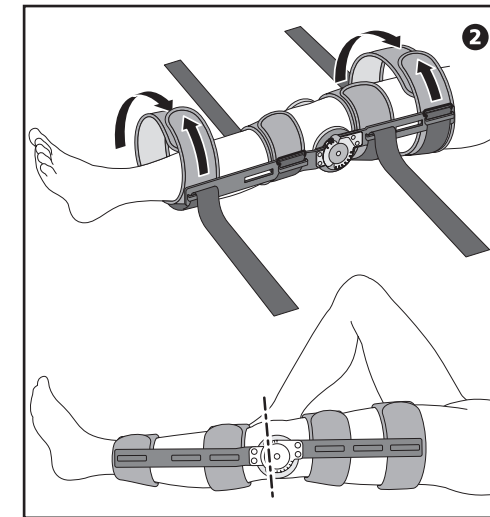
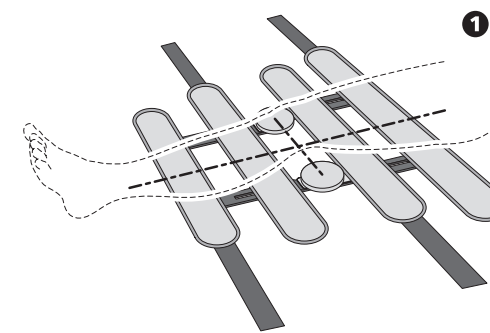
- 1 Open the brace by releasing the straps and foam wraps and laying them out flat. Position the patient's leg (left leg illustrated) so that the knee is centered between the foam hinge pads.
- 2 Wrap the foam wraps around the leg starting with the wraps closest to the knee. The hinges should be positioned slightly above the middle of the knee cap and toward the back of the knee on the sides. If necessary, the side bars may be removed from the foam wraps and repositioned.
- 3 Secure and fasten the 2 straps closest to the knee first. Note that the straps fasten in opposing directions.
- 4 Secure and fasten the remaining 2 straps, also in opposing directions.

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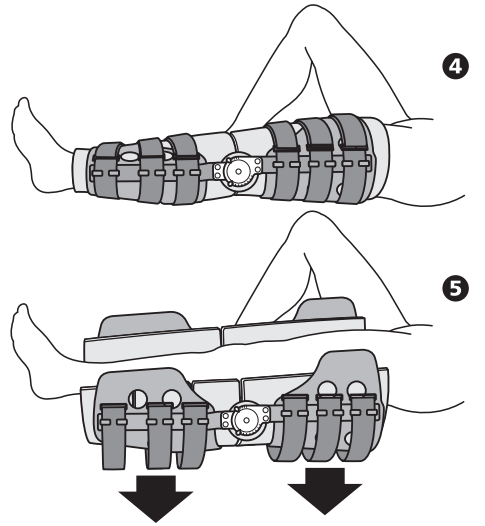
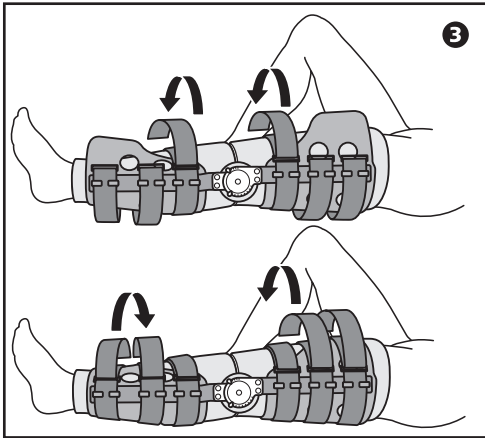
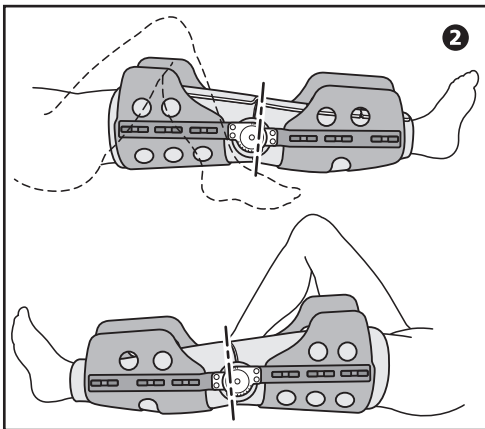
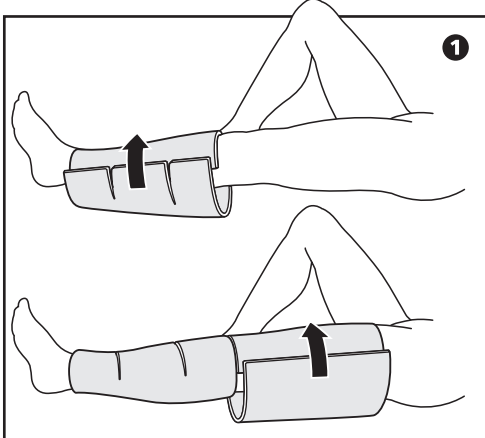
- 1 Wrap and fasten the foam wraps around the leg using the side bar as a gauge for positioning.
- 2 Position and press into place the side bars. Reposition the wraps if necessary to align with the straps. The hinges should be placed slightly above the middle of the knee cap and toward the back of the knee on the sides.
For illustration purposes the straps are not shown in step 2. Do not remove the straps for application.
- 3 Secure and fasten the straps closest to the knee first, then the remaining straps.
- 4 A properly applied POST-OP KNEE BRACE.
- 5 To remove the brace, simply unfasten the straps and open the foam pads. The brace can then be removed and reapplied as a single unit.

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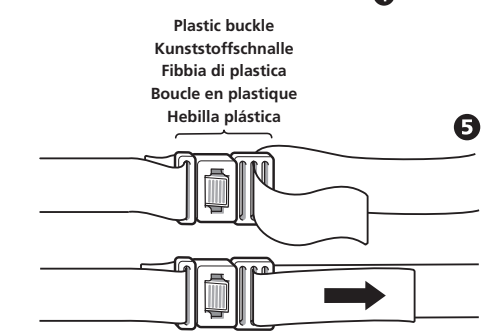
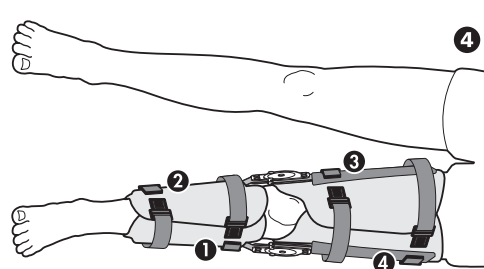
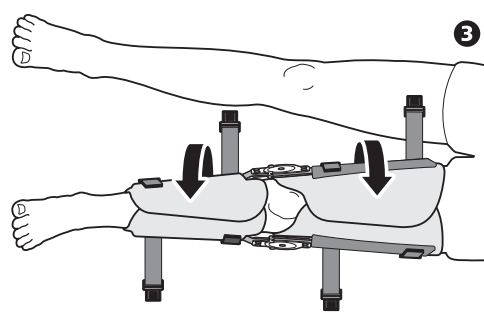
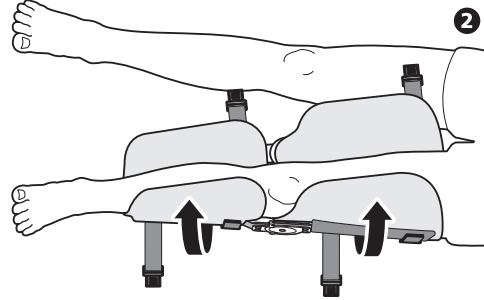
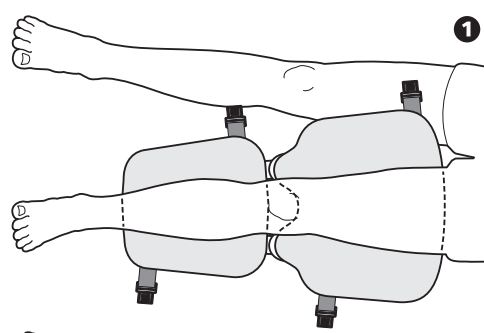
- 1 Open the Postop QuickFit brace and place it under the fully extended leg. The hinges should be at mid-patella level and medial/lateral level of the leg.
- 2 Fold foam wrap over from the lateral or outside of the leg.
- 3 Fold foam wrap over from the medial or inside of the leg so that the wrap fits snugly. Close the wrap at the front of the leg with the hoop and loop closure.
- 4 Buckle the straps, starting with the upper calf (1), the lower calf (2), the lower thigh (3), and last the upper thigh (4).
- 5 Tighten the straps by pulling the strap tight through the buckle and fastening back against itself by using the hook and loop closure.

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TIPS ON USE AND CARE OF YOUR POST-OP KNEE BRACE:

- Close the foam wraps in the front to allow easy removal of the brace later.
- The foam wraps may be trimmed with scissors as needed.
- The side bars may be bent as illustrated at RIGHT to accommodate individual legs. Bend in equal amounts above and below the hinge and on each side.
- Wash foam wraps and straps by hand only and air dry. Extra wraps are available from BREG.
- To tighten straps securely it is easiest to use two hands—one hand to pull the end of the strap after it passes through the ring and the other to hold the ring in place next to the side bar.

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ROM HINGE ADJUSTMENTS:

- 1 Extension limit settings may be selected between 10° of hyperextension and 30° by pulling the tab out and sliding to desired position.
- 2 Flexion limit settings may be selected between 10° of hyperextension and 120°.
- 3 The hinge may be locked by sliding the quick lock button into the locked position at any one of 5 positions: 10° of hyperextension, 0° (neutral), 10°, 20°, 30°.

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