



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ANKLE SUPPORTS

TARGET ANKLE WRAP

- > Features wrap-around design to provide warmth, compression and protection with vel-stretch figure-8 strap for compression over malleolus
- > Indicated for treatment of ankle strains or sprains and athletic protection
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

S/M	L/XL
88-4010	88-4011

SIZING GUIDE

S/M	L/XL
10"-13"	13"-16"

Measurement is circumference from base of heel around ankle

APPLICATION INSTRUCTIONS

1. Open wrap and locate company logo.
2. Place wrap on ankle with logo behind ankle over Achilles tendon area with heel protruding through opening.
3. Attach distal and proximal straps.
4. Grasp both medial and lateral stretch compression straps.
5. Cross straps in front of brace and secure to desired comfort.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.