Instructions for use

Unloader One® Plus

Life Without Limitations®

.....	3
(EN) Instructions for use.....	9
(DE) Gebrauchsanweisung.....	13
(FR) Notice d'utilisation.....	17
(ES) Instrucciones para el uso.....	21
(IT) Istruzioni per l'uso.....	25
(NO) Bruksanvisning.....	29
(DA) Brugsanvisning.....	33
(SV) Bruksanvisning.....	37
(FI) Käyttöohjeet.....	41
(EL) Οδηγίες χρήσης.....	45
(NL) Gebruiksaanwijzing.....	49
(PT) Instruções de Utilização.....	53
(JA) 取扱説明書.....	57
(ZH) 中文说明书.....	60
(KO) 사용 설명서.....	64

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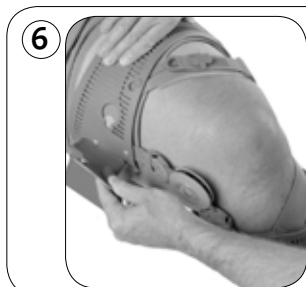
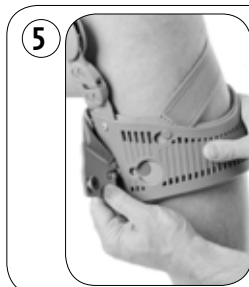
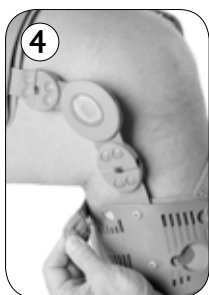
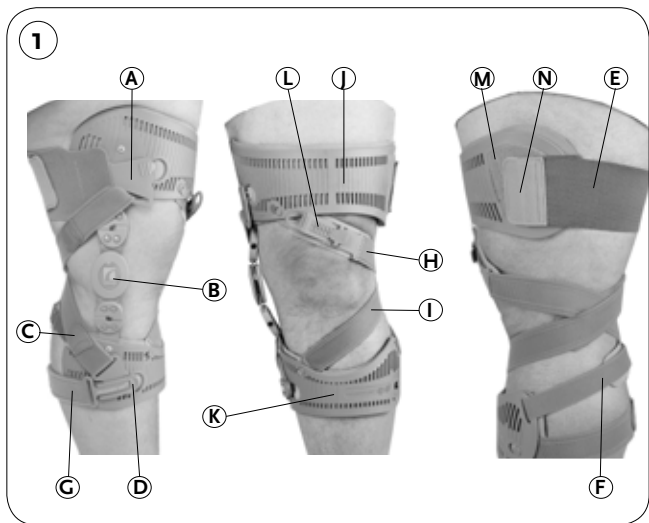
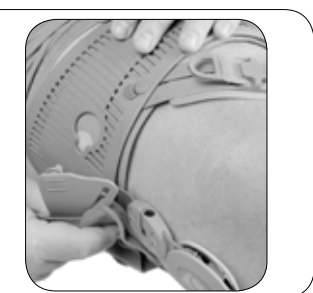
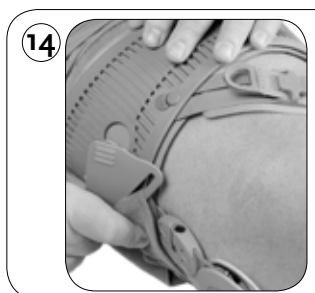
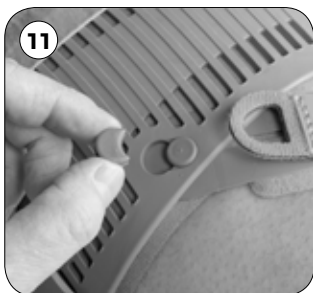
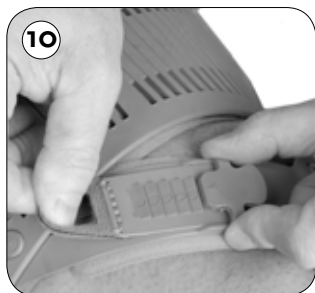
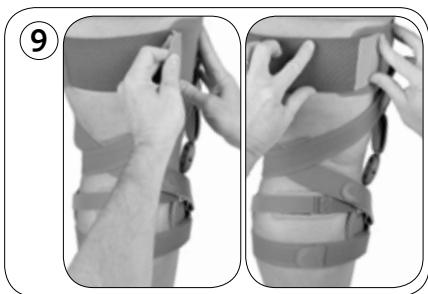
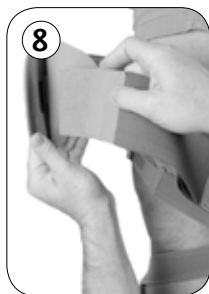
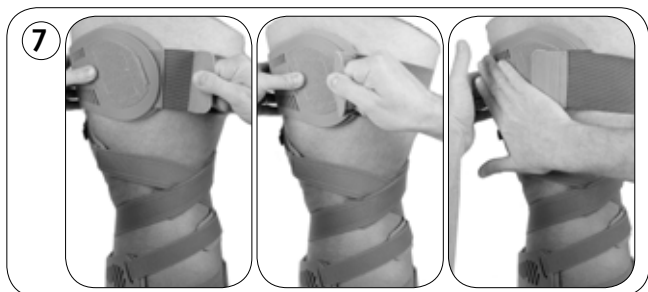
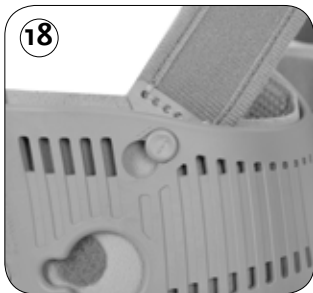
INDICATIONS FOR USE

- Mild to severe unicompartmental osteoarthritis
- Unicompartmental knee conditions that require load reduction to affected compartment such as:
 - Articular cartilage defect repair
 - Meniscal cartilage repair
 - Avascular necrosis
 - Tibia plateau fracture
 - Normal activities of daily living

CAUTION: For patients with peripheral vascular disease, neuropathy, and sensitive skin, close physician supervision is recommended. The Unloader One® Plus is field adaptable to accommodate patient comfort issues. Modifications to flexion or extension stops and shell contour should only be made by the healthcare professional. The healthcare professional is responsible for advising patient on use of the Dynamic Force Control System.

CIRCUMFERENCE LIMITATION

- Thigh circumferences smaller than 18" (45.7 cm) or larger than 33" (83.8 cm)
- Calf circumferences smaller than 14" (35.5 cm) or larger than 24" (61.5 cm)

ADJUSTING HINGE CONTOUR AND CORRECTIVE FORCES

1. Before putting the brace on the leg, loosen the two tamper-proof hinge adjustment screws using the supplied Allen wrench. (Figure 2)
2. Seat the patient on the edge of a chair, leg in full extension.
3. Place brace on leg with center of hinge in line with center of patella. The hinge should be placed midline in the anterior / posterior plane.
4. Insert spacer pads between the patient's leg and the condyle pad of the hinge. (each pad $\approx$ 3 degrees)
5. Align shells and hinge with leg anatomy.
6. With slight pressure against the hinge, tighten the bottom hinge adjustment screw followed by the top adjustment screw. (Figure 3)
7. Remove the brace, tighten the adjustment screws.
8. To ensure adjustment is secure, apply varus or valgus strain to the mechanism. If the mechanism clicks, go to step 1.

INITIAL FIT / DONNING BRACE

1. Ensure upper DFS ratchet is set at one.
2. Align brace on patient's leg with center of hinge at midline of the knee, sagittal (anterior/posterior) plane.
3. Secure gastroc strap by pulling Quick Fit button into calf shell keyhole (match white colors). (Figure 4)
4. Ensure calf Quick Fit buckle is open. Insert the lower Quick Fit buckle (blue) into the blue keyhole on the calf shell. Close the Quick Fit buckle (Figure 5).
5. In the seated position, bend knee to an 80° angle with foot flat on floor.
6. Insert the upper Quick Fit Thigh Connector (yellow) into the yellow keyhole on the thigh shell. (Figure 6)
7. Re-position the integrated strap pad to the location where the two DFS straps cross in the popliteal as needed.
8. Affix the non-hinge end of the elastic thigh strap in proper alignment with the contact closure on the non-hinge end of the thigh shell (Figure 7).
9. Practitioner must make a one-time tension adjustment of the elastic thigh strap (Figure 8). Affix the non-hinge side of the elastic thigh strap to the center of the fastening pad on the shell. To adjust the tension, reposition the opposite end of the thigh strap to the thigh connector fastening pad (Figure 9). Ensure no contact closure hook is exposed to skin.
10. Ensure that the Dynamic Force Control System ratchet is set to one. While knee is flexed to 80 degrees, the practitioner should tighten the upper DFS strap to its proper tension using the contact closure.
11. In the same position of flexion, the practitioner should tighten the lower DFS strap to its proper tension.
12. Adjust the Dynamic Force Control System ratchet on the upper DFS strap to a setting of five. Practitioner will confirm the optimum strap lengths/unloading based on the patient's feedback (Figure 10). Trim straps as needed (see optional ONE-TIME STRAP LENGTH ADJUSTMENT instructions below).
13. If the patient requires more or less unloading, adjust the tension of the upper DFS strap accordingly with Dynamic Force Control System ratchet as needed. (Note: If ratchet has been pulled to its maximum and patient requires additional unloading force, the upper DFS strap must be shortened).
14. Keyhole locks may be inserted into calf and thigh shells (Figure 11) if desired to ensure straps cannot accidentally be disengaged from shells.
15. Ensure strap and strap pad are positioned properly in the popliteal region. (Figure 12)

NOTE: Have patient wear brace for a few minutes to make sure straps are at a comfortable tension.

REMOVING BRACE

1. In the seated position, bend knee to an 80° angle with foot flat on the floor.
2. Press button on Dynamic Force Control System ratchet to release tension on the upper DFS strap to a setting of one (Figure 13).
3. Reduce tension of elastic thigh strap by removing the contact closure from the non-hinge side of the shell. The thigh strap should remain tethered to the shell.
4. Pull upper Quick Fit Thigh Connector out of shell keyhole (Figure 14).
5. Open lower Quick Fit Buckle and remove lower Quick Fit Buckle from keyhole.
6. Remove gastroc strap by detaching Quick Fit button from keyhole.

7. Remove brace from leg.

ONE-TIME STRAP LENGTH ADJUSTMENT (optional)

1. Remove alligator clip and trim straps so they terminate away from the popliteal region. Then replace alligator clip on strap. (Figure 15)
2. Repeat for all straps.

TAMPER-PROOF STRAP LENGTH SET-UP

The brace ships with straps folded externally onto themselves for easy adjustment. If desired, the straps may be reversed and folded with an internal termination for a tamper-proof patient configuration:

1. Remove alligator clip.
2. Pull strap out of D-ring/buckle and thread through D-ring/buckle from outside to inside.
3. Use double-sided hook provided to secure strap end.
4. Repeat for all straps.

RANGE OF MOTION (ROM) ADJUSTMENT

The device comes with a 0° extension stop installed. Additional extension stops of 5°, 10°, and 15° have been provided.

TO CHANGE RANGE OF MOTION

1. Peel condyle pad from hinge.
2. Remove retaining screw.
3. Slide 0° stop out of hinge.
4. Insert desired extension stop.
5. Secure with retaining screw (DO NOT OVER TIGHTEN).
6. Replace condyle pad on hinge.

NOTE: Optimal corrective forces are applied at full extension. If patient requires an extension stop change, make sure to readjust tension in DFS straps if more unloading is required.

BRACE MAINTENANCE

Unloader One® Plus is designed for minimum maintenance and care. Follow these easy maintenance steps to keep the brace in good condition.

CLEANING

- Liners may be wiped down in place with alcohol for a quick cleaning
- For more thorough cleaning, liners, straps and strap pads may be removed
- DO NOT MACHINE WASH
- DO NOT PLACE IN DRYER
- Hand-wash using mild non-detergent soap (maximum 30°C)
- Rinse thoroughly
- Air-dry

NOTE: Strapping sequence is shown in Figures 16. Quick Fit Thigh Connector, Quick Fit buckle, and shells are labeled with matching symbols to ease reassembly (Figures 17-18).

HINGE

- Hinge should only be disassembled by a healthcare professional
- Check regularly for foreign materials (i.e. dirt or grass) in hinge
- Clean dirt from hinge using tap water or compressed air

NOTE: If brace is used in salt water or chlorinated water, rinse well and air dry brace. A regular service routine will ensure hinge, liners, straps and pads are in optimum working order.

BRACE ACCESSORIES

Össur offers accessories that meet specific needs of the Unloader One® Plus. These accessories can be purchased through the healthcare provider or Össur Customer Service Departments:

- Replacement Liners
- Replacement Pad Kit
- AMS Wrap

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Össur Iberia S.A.